


TCU HOOK UP INSTRUCTIONS

✓ THE HOOK UP



<input type="checkbox"/>	SCRUB WITH ALCOHOL THE ELECTRODE SITES ON THE PT. UNTIL CLEAN
<input type="checkbox"/>	SHAVE ELECTRODE SITES ON THE PATIENT
<input type="checkbox"/>	ROUGH ELECTRODE SITES ON THE PATIENT WITH THE PREP PAD
<input type="checkbox"/>	SNAP ON THE LEADS TO THE ELECTRODES
<input type="checkbox"/>	PEEL AND STICK ELECTRODES TO THE PATIENT

✓ PLACEMENT

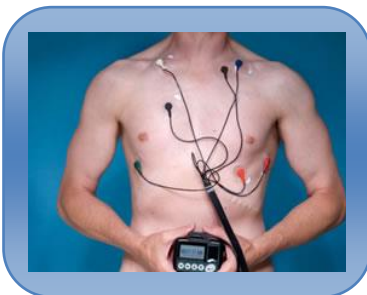

<input type="checkbox"/>	WHITE: RIGHT MANUBRIAL BORDER OF THE STERNUM
<input type="checkbox"/>	BLUE: UPPER HALF OF STERNUM
<input type="checkbox"/>	RED: LEFT ANTERIOR AXILLARY LINE 6 TH RIB
<input type="checkbox"/>	BLACK: LEFT MANUBRIAL BORDER OF THE STERNUM
<input type="checkbox"/>	BROWN: APPROX 1 INCH RIGHT OF XIPHOID PROCESS
<input type="checkbox"/>	ORANGE: LEFT MID CLAVICULAR LINE 6 TH RIB
<input type="checkbox"/>	GREEN: LOWER RIGHT RIB MARGIN OVER MONITOR

✓ MONITOR SETUP



<input type="checkbox"/>	INSERT BATTERY AND COMPACT FLASH CARD INTO THE MONITOR
<input type="checkbox"/>	PRESS THE "ENTER BUTTON"
<input type="checkbox"/>	PRESS THE "RIGHT ARROW KEY" 7 TIMES
<input type="checkbox"/>	PRESS THE "ENTER BUTTON"

✓ PATIENT INSTRUCTIONS



<input type="checkbox"/>	KEEP MONITOR SAFE
<input type="checkbox"/>	NO SHOWERS OR SWIMMING
<input type="checkbox"/>	CONTINUE NORMAL ACTIVITIES
<input type="checkbox"/>	DOCUMENT ANY SYMPTOMS OR STRENUOUS ACTIVITY
<input type="checkbox"/>	PRESS "ENTER BUTTON" FOR SYMPTOMS
<input type="checkbox"/>	RETURN IN 24 HOURS TO HAVE MONITOR REMOVED

CALL **772.370.6490** TO LET TCCM KNOW OF THE HOOK UP. THIS WILL ALLOW FOR BETTER SCHEDULING OF THE PICK UP THE FOLLOWING DAY WHEN THE PATIENT RETURNS.
REMEMBER TO RETURN BATTERIES FOR RECYCLING

